

# RADIESSE®



## Re-Awaken the Skin From Within

Look your best with the  
first and only biostimulator  
approved in the U.S. for  
the face and body<sup>1\*</sup>

Actual patient. Individual results may vary.  
\*Body indications include the hands and décolleté.

### What are RADIESSE® and RADIESSE® (+)?

RADIESSE® and RADIESSE® (+) are dermal fillers that are used for smoothing moderate to severe facial wrinkles and folds, such as nasolabial folds (the creases that extend from the corner of your nose to the corner of your mouth).

RADIESSE® is also used for correcting volume loss in the back of the hands and, when diluted with saline to smooth away wrinkles such as those in the décolleté in patients 22 years of age and older. RADIESSE® (+) is also used for improving moderate to severe loss of jawline contour in adults over 21 years old.

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/RADIESSE (+) **Instructions For Use**.

Targeted vitality for face and body\*

# Own Your Look

RADIESSE® is a proven regenerative biostimulator that firms and tightens skin, reversing visible signs of aging from within.<sup>2-10</sup>

Your provider can tailor your treatments to achieve different goals for your lower face, jawline, hands, and décolleté.<sup>1</sup>

● RADIESSE

● RADIESSE (+)

● RADIESSE 1:2 dilution

● Hands  
(replace loss of  
firmness & volume)

● Skin quality  
(enhance radiance)

● Jawline  
(contour & define)

● Décolleté  
(smooth & revitalize)

● Midface to efface  
the lower face  
(restore shape)

● Nasolabial folds  
(restore shape)

● Moderate to  
severe smile lines  
(sculpt & smooth)

● Sagging jowls  
(reposition & lift)

Actual patient. Individual results may vary.  
\*Body indications include the hands and décolleté.

## RADIESSE® and RADIESSE® (+) IMPORTANT SAFETY INFORMATION

### Who should not use RADIESSE® or RADIESSE® (+)?

You should not use RADIESSE® or RADIESSE® (+) if you have an allergy to any component of the product, if you have a history of severe allergies, if you have a bleeding disorder. You should not use RADIESSE® (+) if you have an allergy to lidocaine or medicines like it.

# RADIESSE®

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/ RADIESSE (+) **Instructions For Use**.

# Activate Skin's Regenerative Power to Bring Out Your Natural Beauty




Actual patient. Individual results may vary.

RADIESSE® uses an ingredient that is also naturally found in your body to regenerate collagen, elastin, and other key proteins.<sup>5,8,10-12</sup>

## FIRMS

Improves skin firmness, elasticity, and hydration<sup>3,5,8,10-13</sup>

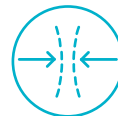

**23x**

more collagen vs placebo<sup>14\*</sup>

## TIGHTENS

Regeneration results in smoother, tighter skin<sup>3,4</sup>



**78%**

of patients showed tighter skin<sup>3†</sup>

## REVERSES

Temporarily reverses visible signs of aging<sup>1,10</sup>



Improvements can last up to 2 years<sup>2</sup>

\*Based on 6-month human tissue study vs placebo.<sup>9</sup>

†Compared to baseline in 20 female subjects with self-evaluated flaccidity scores  $\geq 3$  on a 6-point scale (0, no flaccidity; 5, very severe flaccidity) in zones of interest.<sup>3</sup>

## RADIESSE® and RADIESSE® (+) IMPORTANT SAFETY INFORMATION (cont'd)

### What is the most important information I should know about RADIESSE® and RADIESSE® (+)?

One of the risks with using these products is unintentional injection into a blood vessel. The chances of this happening are very small, but if it does happen, the complications can be serious and may be permanent. These complications, which have been reported for facial injections, can include vision abnormalities, blindness, stroke, temporary scabs, or permanent scarring of the skin. If you have changes in your vision, signs of a stroke (including sudden difficulty speaking, numbness or weakness in your face, arms, or legs, difficulty walking, face drooping, severe headache, dizziness, or confusion), white appearance of the skin, or unusual pain during or shortly after treatment, you should notify your health care practitioner immediately.

**RADIESSE®**

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/RADIESSE (+) **Instructions For Use**.

# Look Your Best at Any Age\*

RADIESSE® is proven to reverse the visible signs of aging.<sup>1,10</sup>

It's no secret: Your skin changes as you get older. Aging and your environment make it harder for your body to produce 2 important components—collagen and elastin—which can hold off lines and wrinkles.<sup>12,15</sup>

Patient at age 25

Patient at age 45



Not an actual patient. Hypothetical representation of skin aging over time.

\*RADIESSE and RADIESSE(+) are approved for use in adults 22 years of age and older.

**RADIESSE®**

## RADIESSE® and RADIESSE® (+) IMPORTANT SAFETY INFORMATION (cont'd)

### What is the most important information I should know about RADIESSE® and RADIESSE® (+)? (cont'd)

As with all procedures that involve an injection through the skin, there is a risk of infection.

Do not use RADIESSE® or RADIESSE® (+) if you have a skin infection until it has healed.

It is not known if RADIESSE® or RADIESSE® (+) is safe or effective in the lips, or in the area around the eyes, or if you are pregnant or breastfeeding.

It is not known if RADIESSE® or RADIESSE® (+) is safe or effective beyond 3 years in the face, 1 year in the hand, and 84 weeks in the décolleté.

Injection of RADIESSE® (+) into the jawline may temporarily alter jaw function.

The safety of RADIESSE® in the décolleté in patients under 22 years of age has not been established.

The microspheres in RADIESSE® and RADIESSE® (+) can be seen in X-rays and CT scans. It is very important that you tell your health care provider that you have had RADIESSE® or RADIESSE® (+) dermal filler. A clinical study found no evidence of RADIESSE® injected in the décolleté being visible on mammograms or breast ultrasounds, but you should still inform your doctor and radiologist that you have received RADIESSE® in the treated area so they can interpret your imaging appropriately.

If you have a history of herpes, you may experience a herpes breakout after receiving RADIESSE® or RADIESSE® (+).

Injection in the back of the hand may result in temporary difficulty performing activities. RADIESSE® may cause nodules, bumps or lumps in the back of the hand and can last up to 1 year.

You should minimize strenuous activity and avoid extensive sun or heat exposure for about 24 hours after treatment and until any swelling or redness has resolved.

### What should I tell my doctor before using RADIESSE® or RADIESSE® (+)?

Tell your health care provider if you are taking blood thinners or medicines that can interfere with the clotting of blood, such as aspirin or warfarin. These medicines might make it more likely that you will experience bruising or bleeding at the injection site. Tell your health care provider if you have any diseases, injuries or disabilities of the hand, if you have a history forming large, raised scars or if you have had any other skin treatments such as skin peels.

### What are the most common adverse events with RADIESSE® or RADIESSE® (+)?

The most common adverse events seen in clinical studies of RADIESSE® used in the hands include bruising, redness, swelling, pain, itching, nodules or bumps/lumps, difficulty performing activities, loss of sensation and other local side effects. The most common adverse events seen in clinical studies of RADIESSE® or RADIESSE® (+) used in the face include bruising, redness, swelling, pain, itching, difficulty chewing, and other local side effects.

Delayed-onset inflammation near the site of injection is one of the known adverse events associated with dermal fillers. Cases of delayed-onset inflammation have been reported to occur at the treatment site following viral or bacterial illnesses or infections, vaccinations, or dental procedures. Typically, the reported inflammation was responsive to treatment or resolved on its own.

These are not all of the possible side effects with RADIESSE® or RADIESSE® (+). Merz collects information about adverse events seen with RADIESSE® and RADIESSE® (+) outside of clinical studies. These events are included in the RADIESSE® and RADIESSE® (+) Patient Information Guide based on an assessment of seriousness and potential causal relationship to RADIESSE® or RADIESSE® (+). Please see the Patient Information Guide available at [www.radiesse.com](http://www.radiesse.com) for list of these events. Tell your health care provider about any side effects that bother you or do not go away.

Important: For full safety information, please visit [www.Radiesse.com](http://www.Radiesse.com) or call MyMerz Solutions at 1-844-469-6379.

RADIESSE® and RADIESSE® (+) are available by prescription only.

# Rely on RADIESSE® for Younger-Looking Skin

Delivers natural-looking results that last up to 2 years.<sup>2</sup>



**8 in 10 patients felt their appearance was improved today compared to their appearance before their RADIESSE treatment.<sup>14</sup>**

## How to Get Started

**RADIESSE treatments are generally completed in a brief in-office visit by a trained healthcare provider.**

RADIESSE helps smooth out lines and wrinkles, improving skin quality gradually over time.

- To get more personalized information, schedule a consultation with a health care provider
- Visit [RADIESSE.com](https://www.RADIESSE.com) to find a provider near you

**References:** **1.** Radiesse [Instructions for Use]. Franksville, WI: Merz North America, Inc; 2026. **2.** Bass LS, et al. *Aesthet Surg J*. 2010;30(2):235-238. **3.** Cogorno Wasyilkowski V. *Clin Cosmet Investig Dermatol*. 2015;8:267-273. **4.** Draoui O. Chapter 1: The Story of Calcium Hydroxylapatite. In: van Loghem J, ed. Calcium Hydroxylapatite Soft Tissue fillers, *Expert Treatment Techniques*. 1st ed. Taylor & Francis Group; 2020. **5.** González N, et al. *Dermatol Surg*. 2019;45(4):547-551. **6.** Kim J. *Clin Cosmet Investig Dermatol*. 2019;12:771-784. **7.** Silvers SL, et al. *Plast Reconstr Surg*. 2006;118(3 suppl):34S-45S. **8.** Yutskovskaya YA, et al. *J Drugs Dermatol*. 2017;16(1):68-74. **9.** Moradi A, et al. *J Drugs Dermatol*. 2021;20(11):1231-1238. **10.** Aguilera SB, et al. *Aesthet Surg J*. 2023;43(10):1063-1090. **11.** Yutskovskaya YA, et al. *J Drugs Dermatol*. 2020;19(4):405-411. **12.** Yutskovskaya Y, et al. *J Drugs Dermatol*. 2014;13(9):1047-1052. **13.** Palo JS, et al. *Skin Res Technol*. 2024;30(7):e13835. **14.** Data on file. Merz North America, Inc. 2025. **15.** Baumann L. *J Pathol*. 2007;211(2):241-251.

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](https://www.RADIESSE.com) for RADIESSE/ RADIESSE (+) **Instructions For Use**.

**RADIESSE**®

MERZ AESTHETICS®

©2026 Merz North America, Inc. All rights reserved. MERZ AESTHETICS is a registered trademark of Merz Pharma GmbH & Co. KGaA in the U.S. RADIESSE is a registered trademark, and the R Logo is a pending trademark, of Merz North America, Inc. in the U.S. April 2026 US-RAD-2500160